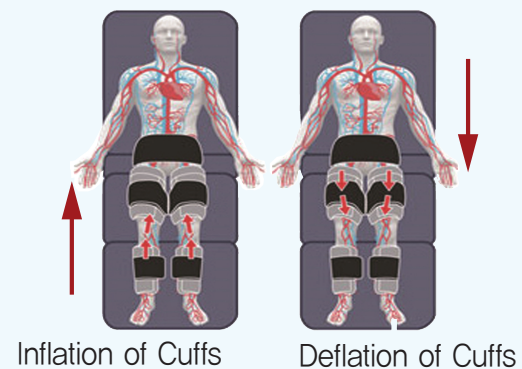
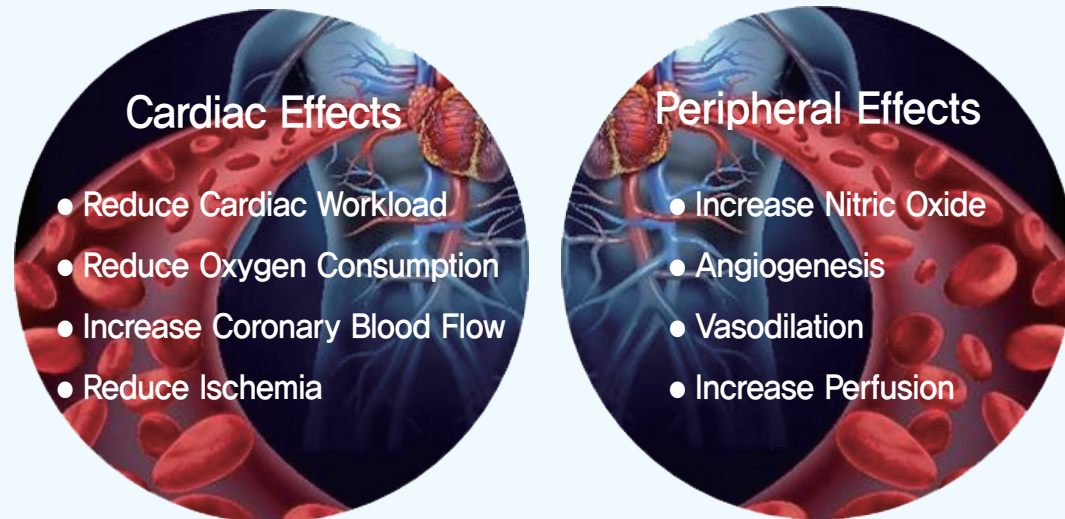
External Counterpulsation Device 체외 역박동 치료기 (ECP)

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• Renew™ NCP-5 Functionality

- Automatic firmware detection of the QRS complex and the R-wave.
- Automatic inflate / deflate start time setting ensures that counterpulsation occurs only during diastole. A push button switch allows manual fine-adjustment of the inflation / deflation times.
- Notch filters and band-pass filters eliminate electrical interference to the cardiac input signal.
- Software options provide ability to review treatment data or individual ECG events that occurred during treatment or previous treatment sessions.

• Software Safety Features

- System will be alerted and inflation / deflation will be suspended in any of these events :
 - If heart rate exceeds the normal range of operation. (<36 BPM or >124 BPM)
 - When an external (patient) ECG signal is not available or reliable.
 - If the laptop is unable to display data.

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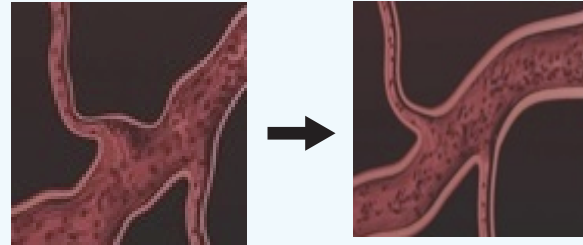
■ FEATURES (Renew™ NCP-5)

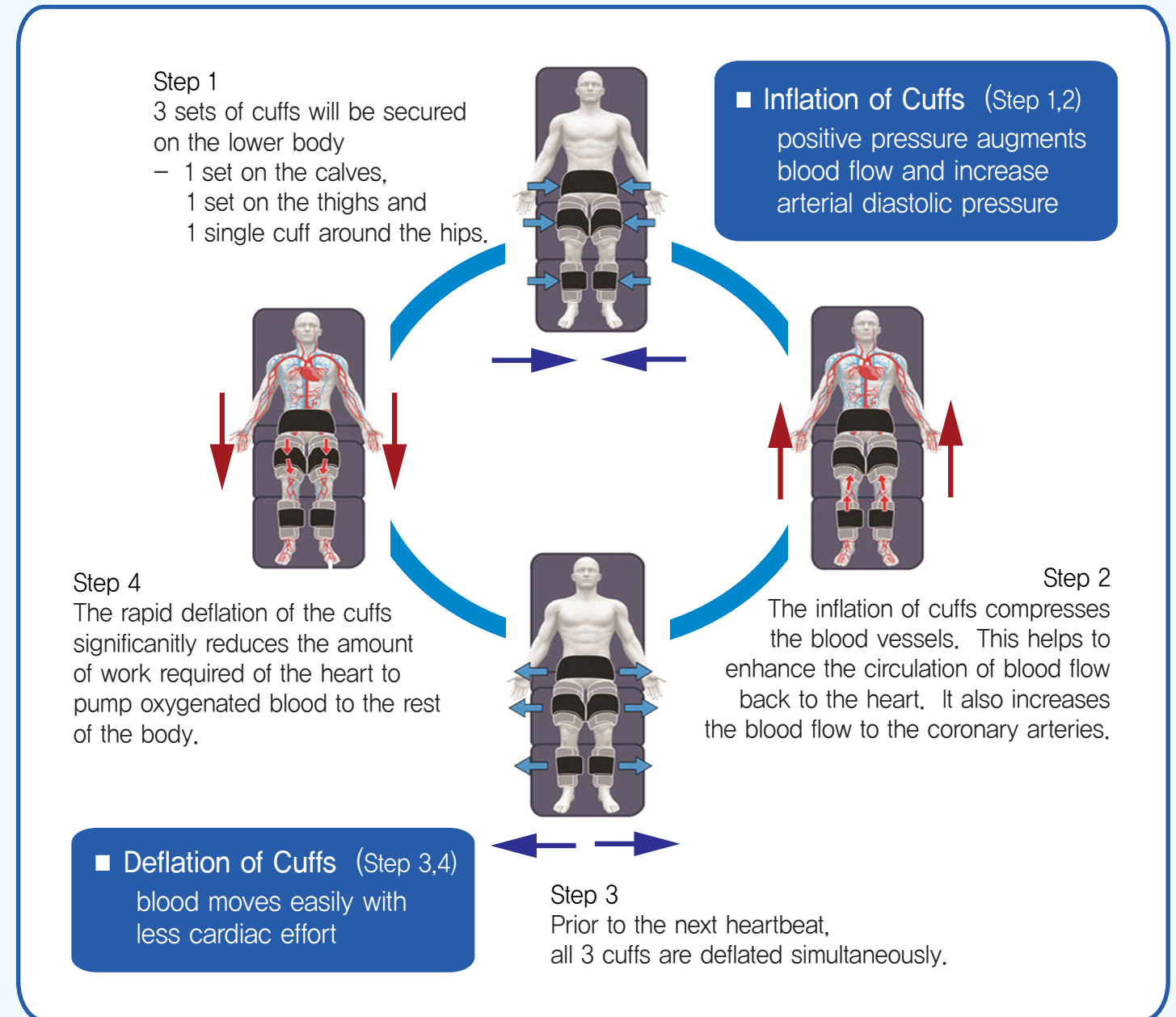
- ▶ Foldable and Compact
Lightweight Renew NCP-5 unit that folds for compact storage and ease of transport from one treatment area to another.
- ▶ Superior Comfort
Comfortable treatment platform which uses memory foam mattress to ensure firm support throughout Renew ECP Therapy.

- ▶ Quiet
Reduced noise during operation improved air compressor system.
- ▶ Simple user interface
Functionally and practically designed with a simple user interface for easy adjustment.
- ▶ Built-in Laptop
A built-in laptop mounted on a rotatable control console.

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
Pulse Waveform Change Monitoring

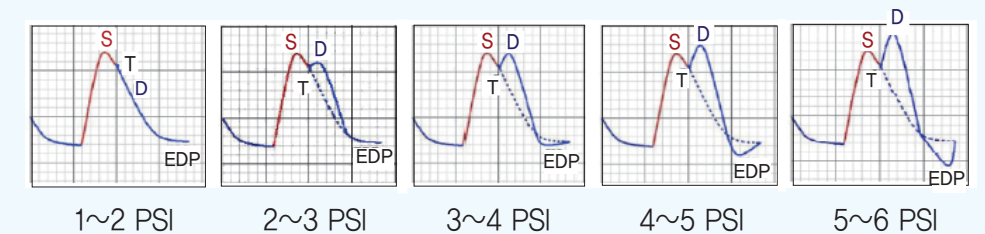Straib change after systole (postsystolic strain index : PSI)

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